




Stoma
Skin
Soul



A range of applications
for the Dansac **TRE™ seal**

Multiple Positioning

- Providing protection and security where needed
- Can be used to avoid contact with mid-line wound in post operative setting
- Rotate seal to provide more barrier where skin folds or retracts
- Enables optimum fit between stoma and skin

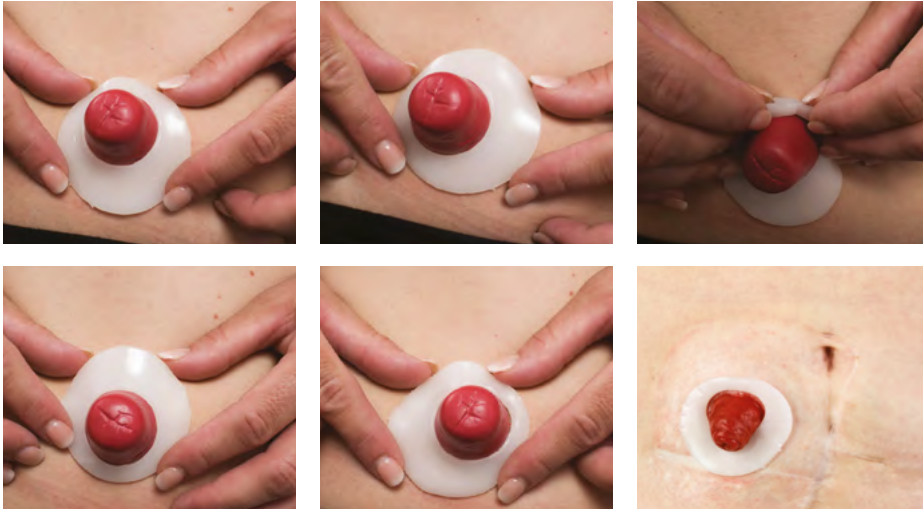

Showing how to place the TRE™ seal to avoid contact with the mid-line wound

Wrap Around



- Can be used where the stoma end is larger than the base e.g. loop stomas
- Break open the seal at the narrowest part, and apply around the base of the stoma ensuring the ends overlap (see images)
- Close positioning of the seal at the mucocutaneous junction can assist with the protection of peristomal skin

Mucocutaneous junction - where your stoma meets the skin
Peristomal skin - area of skin around your stoma

Range of Applications

- Compatible with all stoma bags
- Can be applied directly on to the skin barrier or peristomal skin



Application to the peristomal skin

Additional Applications

- Can be stretched and moulded to form custom shapes
- Fills uneven skin surfaces around the stoma
- Helps improve the fit of the skin barrier



Three levels of protection

Sealing in Skin Health

Dansac TRE™ seal is more than just a seal.

With three levels of protection, the Dansac TRE seal has been designed to help keep skin naturally healthy.



pH balance

Designed to help manage the skin-damaging effects of digestive enzyme activity



Absorption

Helps absorb excess moisture to maintain skin's natural balance without the seal losing internal or external strength



Adhesion

Designed to provide a secure, flexible seal to protect the skin from stoma fluid and to be easy to remove



Code	Units	Hole diameter	Seal diameter
071-30	30/box	30 mm	57.5 mm
071-40		40 mm	70 mm



Code	Units	Hole diameter	Seal diameter
072-48	10/box	18 mm	48 mm
072-98	3/box	18 mm	98 mm
072-57	10/box	27mm	57mm

NEW

Dansac Australia

PO Box 375
Box Hill, VIC 3128
1800 880 840

www.dansac.com.au

Dansac New Zealand

PO Box 107097
Auckland Airport, Auckland 2150
0800 678 669

www.dansac.co.nz

www.facebook.com/dansacostomy [@dansac_ostomy](https://www.instagram.com/dansac_ostomy)

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

Dansac, the Dansac logo and TRE are trademarks of Dansac A/S. All other trademarks and copyrights are the property of their respective owners. © 2024 Dansac A/S. AUD163. April 2024.

dansac